

PACS SHUTDOWN AND STARTUP PROCEDURES

Purpose: To provide boot up and shutdown procedures for the various components of the Picture Archiving and Communications System (PACS) **Procedure:** Use the steps listed to boot up, shutdown, or reboot PACS components. If problem is not corrected call for service. During regular business hours issues can be reported to PACS administrator, Courtney Cooper at 56454. During after hours, weekends, and holidays PACS issues can be called into GE CARES at (800) 437-1171, system ID: 318675PVS

Centricity Workstation

- **Boot up**
 1. Push right on button on the computer – check to see if Monitor is turned on
 2. System will boot “Window NT” and automatically start Radiograph Review Application
 3. Login
- **Shutdown**
 1. Logout of Radiograph Review Application
 2. Choose quit at login prompt, then yes
 3. Click Start in bottom left corner of screen and choose Shutdown
- **Tricks** – If System locks up – you can always hit Ctrl + Alt + Del keys to end process which was locked up. Re-start Radiograph Review Application – click Start, Program, Radiograph Review

Reviewed: 12/17/06

Revised : 01/22/08

Reviewed: 01/09

Revised: 03/10/11