

Routine Head 1.1

LightSpeed16

Indications: Headache, syncope, CVA, ICH, AMS, Hydrocephalus

Use Contrast: Tumor, abscess, infection, metastasis, abnormal non-contrast CT

Patient Preparation: None

Group 1: Base of skull through vertex

Scan Mode	Thickness Speed Pitch	Table Interval	SFOV	kV	Auto mA/ Noise Index	Prep Time (sec)	Recon Type
1.0 sec Axial Full	5 2i	5	Head	120	380 2.8	N	Standard

Algorithm: Standard

Network: PACS.