

Routine Sinus 2.1 LightSpeed16

Indication: Sinusitis, Face pain, Tumor

Use Contrast: If clinically indicated

Patient Preparation: None

Group 1:

Scan Mode	Thickness Speed Pitch	Table Interval	SFOV	kV	Auto mA/ Noise Index	Prep Time (sec)	Recon Type
1.0 sec Helical Full	2.5 0.562 11.25	2.5	Head 15cm	120	380 8	N	Bone Standard

Algorithm: Standard and bone. Show recon 1 Standard 2 Bone 3 Standard

Network: Recon 1 and 2 to PACS.

Written: 07/06

Revised: 01/2009